

1" scale


Intubation

Procedure

Neck Guard

FOLD

*Join to other piece of pattern*

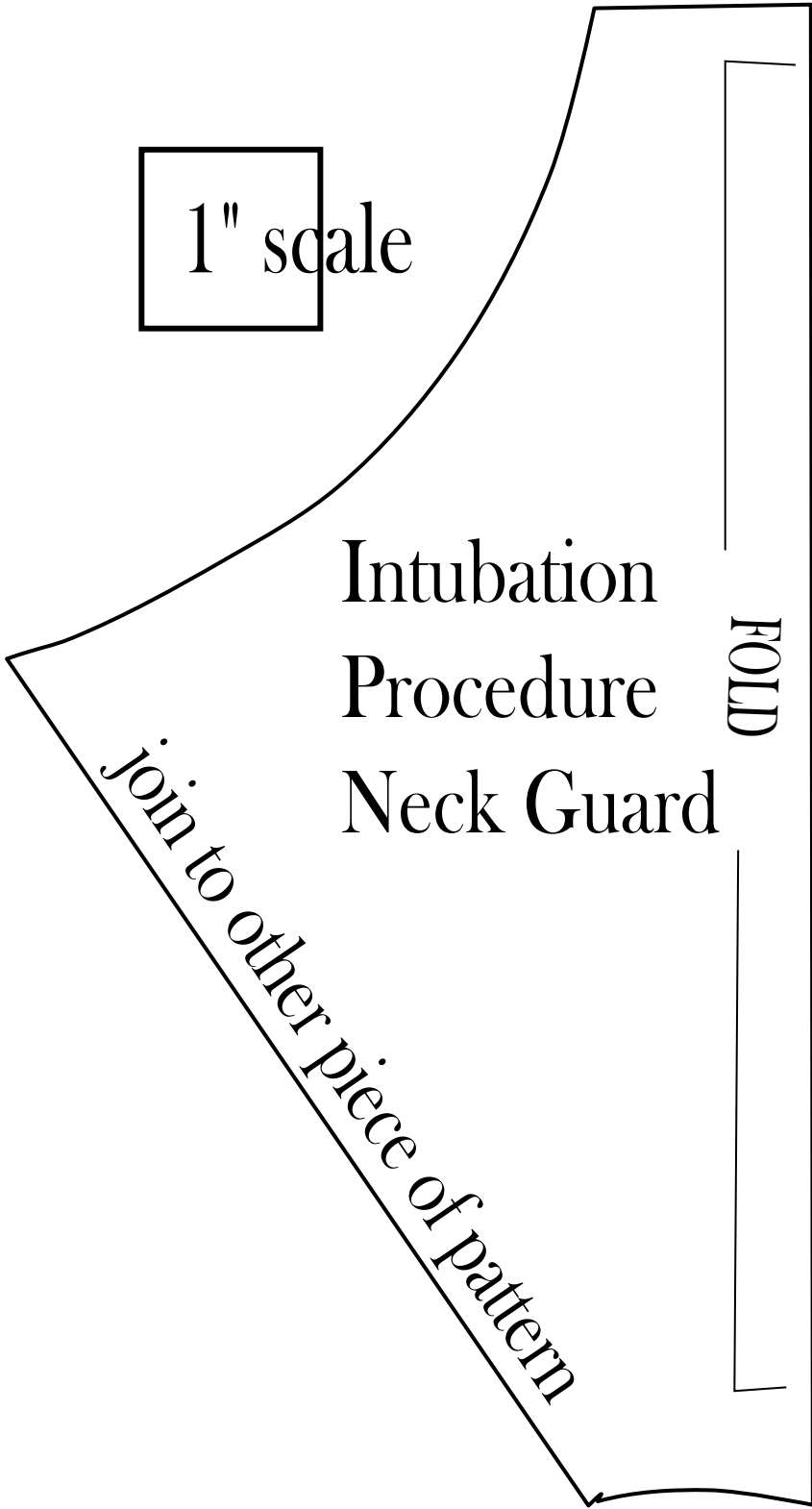

1" scale

cut one on fold

cut 4@ 2 7/8" x 14 (ties)

Cut 1 @ 2 7/8" x 10 (binding for chin)

1. fold binding strip in half and press, wrong sides together and open, then fold raw edges toward middle to create binding
2. Pin along chin edging and stitch in place making sure to catch both inside and outside edges - a small zigzag can be used
3. turn in 1/4" and turn again along all remaining edges and stitch along the edges.
4. Press tie strips in the same manner as the binding and stitch along the entire length
5. stitch into place where noted on pattern

*Join to other piece of pattern*

*chin edge*

tie placement

